

How to apply Dansac TRE™ seal



Dansac TRE™ technology has received official dermatological accreditation from the Skin Health Alliance, which may bring confidence and reassurance to those who use our products.

dansac

How to apply TRE seal



For further information or samples about our pouches, visit us at www.dansac.co.uk

WARNINGS: Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage. Ensure proper fit as instructed. If the barrier opening is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time.

NOTES: Please note that multiple individuals are represented on this material for the application of the product. To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion. Product fit is affected by many factors such as stoma construction, stoma output, skin condition, and abdominal contours. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or local distributor.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.

Dansac, the Dansac logo and TRE are trademarks of Dansac A/S. All other trademarks and copyrights are the property of their respective owners. © 2023 Dansac A/S

